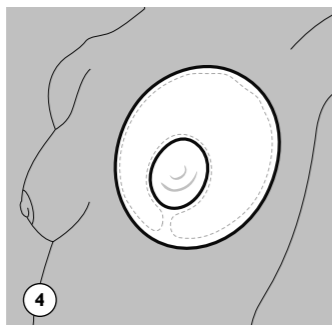
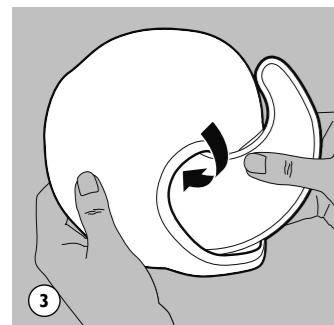
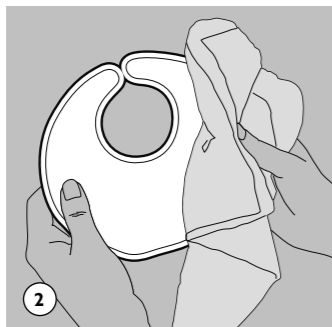
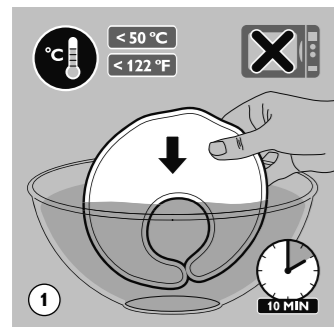
AVENT

A

GB

**Note:** Wash the sleeve before first use. Before you apply the thermopad to the breast, make sure its temperature is acceptable by testing it on the back of your hand. Always use the thermopad together with the sleeve provided.

**A Warm use: to stimulate milk flow before breastfeeding**  
 • Immerse the thermopad in a bowl of hot water and let it warm to the optimum temperature (Fig. 1).  
 • Dry the thermopad with a cloth or towel (Fig. 2).  
 • Insert the thermopad into the sleeve supplied (Fig. 3).  
 • Apply the thermopad to the breast (Fig. 4).

**B Cold use: to soothe sore or engorged breasts**  
 • Place the thermopad in the refrigerator or freezer and let it cool to the optimum temperature for 24 hours (Fig. 5).  
 • Alternatively, immerse the thermopad in a bowl of ice water (Fig. 6).  
 • Insert the thermopad into the sleeve supplied (Fig. 7).  
 • Apply the thermopad to the breast (Fig. 8).

**Cleaning and storage**  
 Before you use the thermopad for the first time and after each use, gently clean the pad in warm water with some washing-up liquid. Do not use a brush, scourer or abrasive cleaning agent. • You can clean the sleeve by hand or in the washing machine at a low temperature. • Store the thermopad and sleeve in a cool, dry place out of direct sunlight and out of the reach of children.

**Safety tips and troubleshooting**  
 Do not heat the thermopad in a microwave. • Do not use the thermopad without the sleeve provided. Never put the frozen thermopad directly onto your skin. • Do not use the thermopad on damaged skin. • Do not use the thermopad while you are feeding your baby or while expressing milk with a breast pump. • If the thermopad becomes damaged or appears to be leaking, stop using it and throw it away. • Do not swallow the contents of the thermopad. • After freezing the gel inside the thermopad has a snow-like appearance. This is normal. The gel in the pad returns to its normal condition in a few hours. • If you have any concerns about using the thermopad, please consult your doctor or midwife.

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Philips AVENT, a business of Philips Electronics UK Ltd. [www.philips.com/AVENT](http://www.philips.com/AVENT)  
 Philips AVENT, a business of Philips Electronics UK Ltd. Surrey, GU2 8XH  
 Trademarks owned by the Philips Group. © Koninklijke Philips Electronics N.V. All Rights Reserved.

- GB call free on 0800 331 6015
- IE call free on 1-800-500-000
- FR 0800 080 190
- DE 0800 000 7520 (kostenfrei)
- ES 900 800 655 (Teléfono local gratuito)
- IT 800 088774 (Numero gratuito - il vostro operatore telefonico potrebbe addebitarvene i costi della chiamata)
- DK 06-1-363-2903
- PL 00800 311 1318
- PT 800 20 47 23
- RU 8 800-200-08-80; 8 800-200-08-83 (звонок-услуга по вопросам грудного вскармливания)
- SE 0800 261 32 02
- SI +386 1548 4300
- SK 0800 261 32 02
- TR 400 880 0008
- TC +886 (2) 8245 7399 / 0800-231-0996
- AR 0800 800 190
- HR 01 336 9520
- HU 06-1-363-2903
- CA 1 800-54-AVENT
- MX 1 800-54-AVENT

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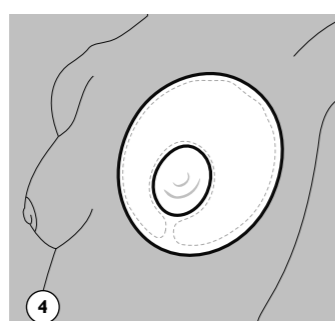
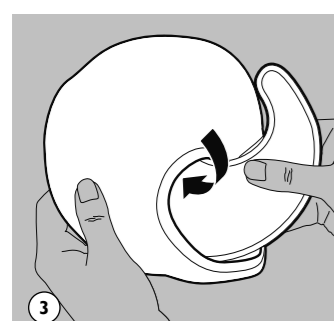
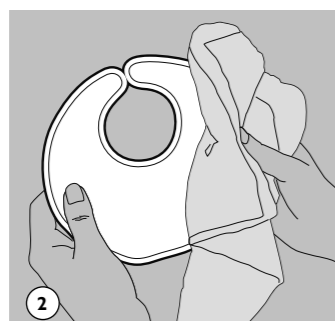
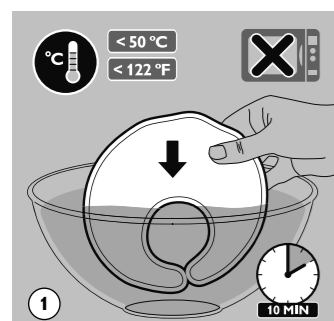
AVENT

Philips AVENT, a business of Philips Electronics UK Ltd.  
 Philips Centre, Guildford Business Park, Guildford,  
 Surrey, GU2 8XH  
 Trademarks owned by the Philips Group.  
 © Koninklijke Philips Electronics N.V.  
 All Rights Reserved.

www.philips.com/AVENT

CH	056 26 56 56	call free on	0800 331 6015
DK	02-709-1200	call free on	1800 744 5477
FR	0800-073 0076		
GB	02-709-1200		
GR	02-709-1200		
HR	0800 073 0076		
IE	0800 073 0076		
IT	800 20 47 23		
JP	0800 073 0076		
KR	0800 073 0076		
LT	0800 073 0076		
LU	0800 073 0076		
NL	800 20 47 23		
NO	0800 073 0076		
PL	0800 073 0076		
PT	800 20 47 23		
RU	8 800-200-08-80		
SE	0800 073 0076		
SI	0800 073 0076		
SK	0800 073 0076		
TR	0800 073 0076		
US	1 800 54 AVENT		
CA	1 800 54 AVENT		
MX	1 800 54 AVENT		

A

**GB**  
 Note: Wash the sleeve before first use.  
**Before you apply the thermopad to the breast, make sure its temperature is acceptable by testing it on the back of your hand. Always use the thermopad together with the sleeve provided.**

**A Warm use: to stimulate milk flow before breastfeeding**

- Immerse the thermopad in a bowl of hot water and let it warm to the optimum temperature (Fig. 1).
- Dry the thermopad with a cloth or towel (Fig. 2).
- Insert the thermopad into the sleeve supplied (Fig. 3).
- Apply the thermopad to the breast (Fig. 4).

**B Cold use: to soothe sore or engorged breasts**

- Place the thermopad in the refrigerator or freezer and let it cool to the optimum temperature for 24 hours (Fig. 5).
- Alternatively, immerse the thermopad in a bowl of ice water (Fig. 6).
- Insert the thermopad into the sleeve supplied (Fig. 7).
- Apply the thermopad to the breast (Fig. 8).

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**Safety tips and troubleshooting**  
 Do not heat the thermopad in a microwave.  
 • Do not use the thermopad without the sleeve provided. Never put the frozen thermopad directly onto your skin. • Do not use the thermopad on damaged skin. • Do not use the thermopad while you are feeding your baby or while expressing milk with a breast pump. • If the thermopad becomes damaged or appears to be leaking, stop using it and throw it away. • Do not swallow the contents of the thermopad. • After freezing the gel inside the thermopad has a snow-like appearance. This is normal. The gel in the pad returns to its normal condition in a few hours. • If you have any concerns about using the thermopad, please consult your doctor or midwife.

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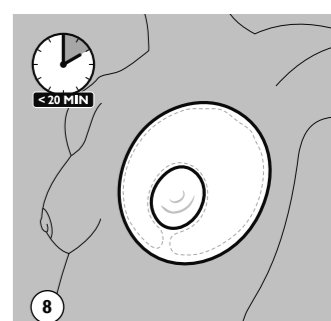
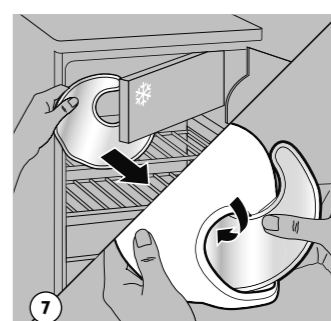
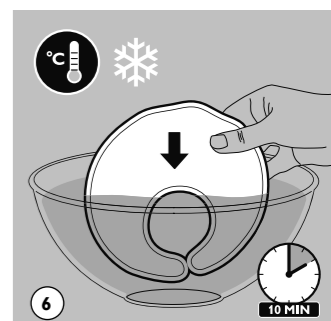
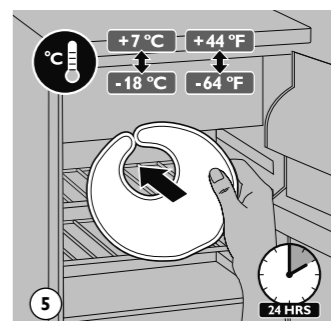
B

**US**  
 Note: Wash the sleeve before first use.  
**Before you apply the thermopad to the breast, make sure its temperature is acceptable by testing it on the back of your hand. Always use the thermopad together with the sleeve provided.**

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**Safety tips and troubleshooting**  
 Do not heat the thermopad in a microwave.  
 • Do not use the thermopad without the sleeve provided. Never put the frozen thermopad directly onto your skin. • Do not use the thermopad while you are feeding your baby or while expressing milk with a breast pump. • If the thermopad becomes damaged or appears to be leaking, stop using it and throw it away. • Do not swallow the contents of the thermopad. • After freezing the gel inside the thermopad has a snow-like appearance. This is normal. The gel in the pad returns to its normal condition in a few hours. • If you have any concerns about using the thermopad, please consult your doctor or midwife.

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